



INTERNATIONAL
PRODUCT CATALOG



NEODRYS®

NEODRYS®

Stop wasting money on pads and rolls that either fall apart or get saturated quickly.
When you add up the reduction in botched procedures and wasted time - **NEODRYS cost less!**



Absorb

- Super absorbent core—out performs messy cotton rolls and other absorbent pads.
- No messy leaks, even with the core fully saturated.


Protect

- Protects procedures from moisture contamination from the parotid gland for up to 15 minutes.
- Stays in place yet removes easily with water spray.
- Protective backing plus poly stiffener provide cheek retraction and protection.

Reflect

- Choice of Original white or Reflective backing brightens the oral cavity, while retracting the cheek for an improved view.

NeoDrys Features

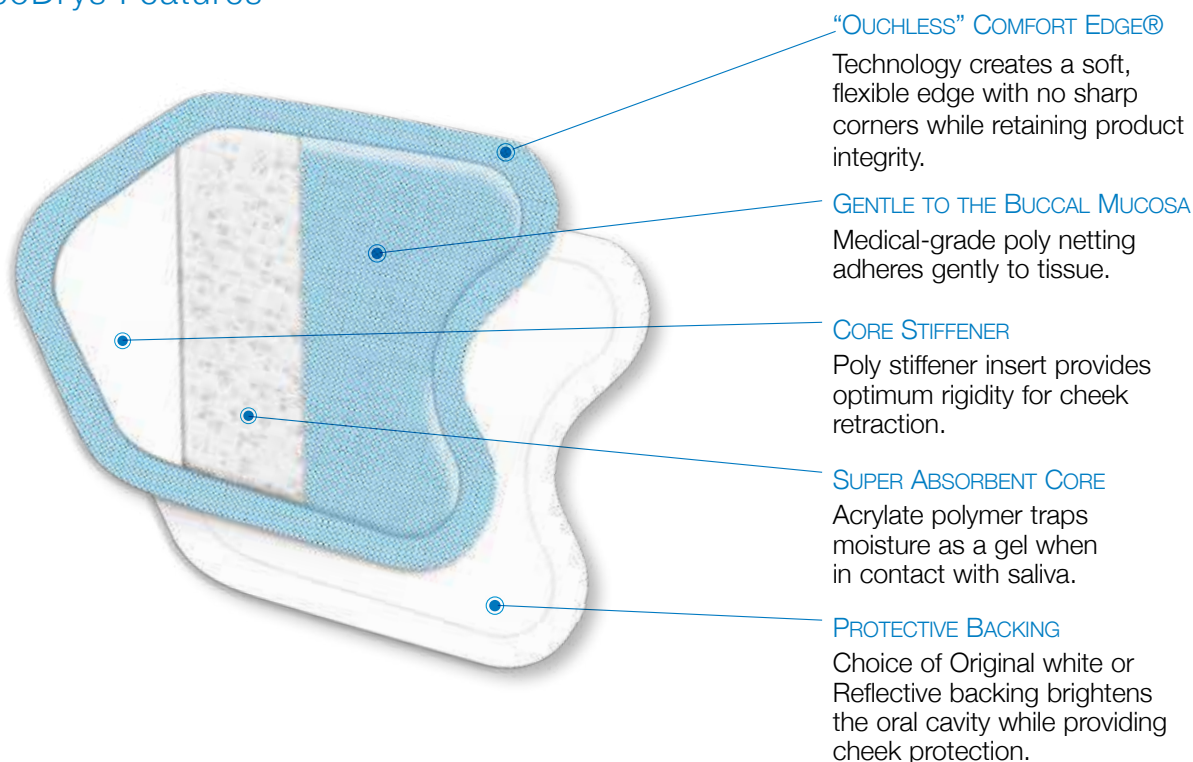


Patents:
US7425664
EP1676552

Method of Application


Select NEODRYS size which adequately covers the buccal mucosa. Insert as shown with color side against cheek and point to back of mouth. In a few seconds, NEODRYS will begin to adhere to the tissue and stay in place.

Important: To remove without irritating tissue, release adhesion with ample water spray to the buccal side of the NEODRYS.


Color to Cheek

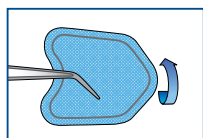

Point to Back

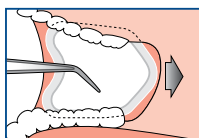

Spray to Release

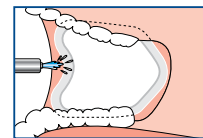

ABSORB, PROTECT & REFLECT

Easier & More Effective Treatments.

- Sealants
- Composites
- Cementation
- Impressions
- Occlusion Marking
- Orthodontics
- CAD/CAM Prep
- Amalgam Preps
- Crown & Bridge
- Laser Dentistry
- Fluoride Applications
- Core Buildups

Color-Coded Sizes

NEODRYS are color coded for quick identification. Available in two sizes: Large (blue) and Small (yellow). Packaged in convenient, easy-to-handle packs of 50.



Choice of Backing

Available in Reflective light-enhancing, mirror-like finish or original white backing.

White Backing



50 per pack

Order No.

DYS - Small (Yellow)
DYL - Large (Blue)


Reflective Backing



50 per pack

Order No.

DYRS - Small (Yellow)
DYRL - Large (Blue)

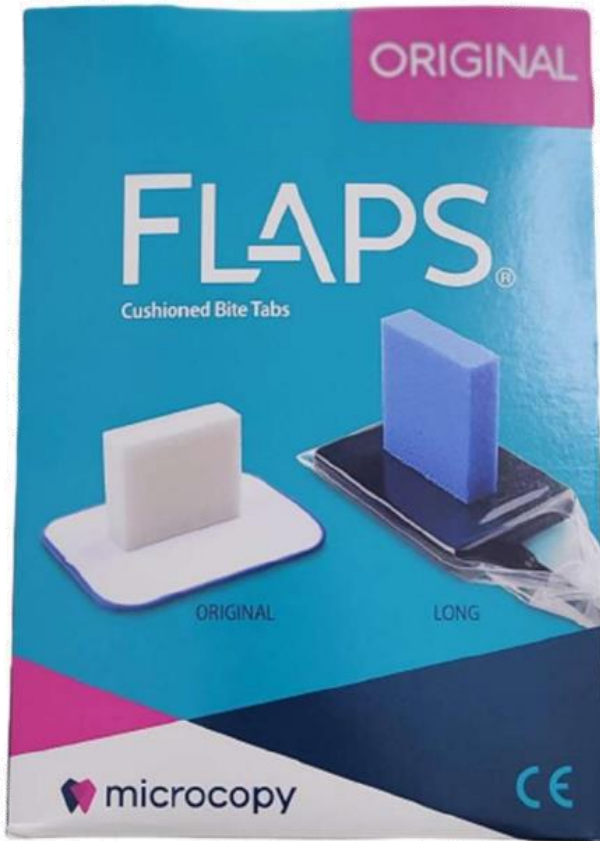

FLAPS®

Simple as that!

Self-sticking foam tabs attach quickly and are easy to position.

We're on point


Cushioned tab holds film or sensor securely, allows full occlusion and provides visual reference for accurate placement and alignment.

Tooth pillows?

Soft cushioning without hard surfaces, provides the ultimate in comfort and small size reduces gagging.

Kick 'em to curb

Use-and-toss convenience—extremely cost effective too!

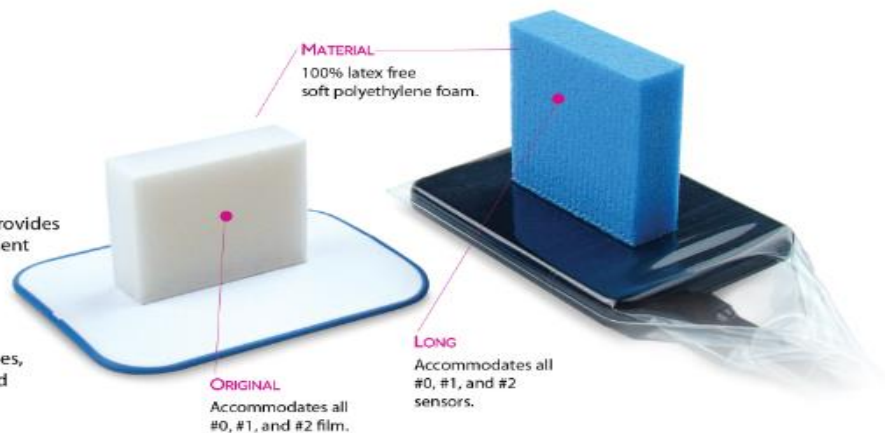

"Flaps are very easy, simple, and fast to use. Plus, patients don't gag like they do with other devices."

Tamara Simac RDA



ORIGINAL (WHITE)

500 per pack



LONG (BLUE)

375 per pack



AL THANAYA
PHARMACEUTICALS LLC